

Proper way to push a Broda Chair.

Elite Models (35V, 85V and 785):

These have two (2) Multi-Directional Casters. Above the back casters you will see a piece sticking out beyond the casters – we call this the “Kicker”. This device is used to Lock, Swivel or allows multiple maneuvering.

1. 1. When you want to go down a hall and you don't want the chair to veer all over the place, position the “kicker” into the Directional lock position (Pushed down towards the back). This will keep the chair in a straight line.
2. 2. When you need to move the chair diagonally or sideways, position the “Kicker” into the Multi position (Horizontal) and this will enable you to move the chair in any direction you need.
3. 3. When you need to lock the chair to do transfers, etc., position the “Kicker” into the Lock position (Pushed down towards the front) and this will Lock the chair.

Basic Models (30VT, 80V, MID and Shower):

These chairs have one (1) Directional Locking Caster. At the rear of the chair you will see one caster that is a different colour - we call it the “Shoe”. (This is the Directional Lock Caster)

1. 1. When you want to go down a hall and you don't want the chair to veer all over the place, position the “Shoe” into the Directional lock position (Pushed down towards the back). This will keep the chair in a straight line.
2. 2. When you need to move the chair diagonally or sideways, position the “Shoe” into the Multi position (Pushed Up) and this will enable you to move the chair in any direction you need.
3. 3. When you need to lock the chair to do transfers, etc., The other three (3) casters will lock.

***NOTE:** When pushing any chair, have the chair Tilted (if possible) so that you can have your hands at a comfortable height and try to position your hands as far out on the chair as possible to give you better leverage.